# HUSKY Aluminum Mobility Carrier Installation Instructions


## SPECIFICATIONS

- Mobility Carrier Dimensions:**
- Deck size: 50-1/2" L x 30" W x 4" H
  - Ramp size: 47" L x 29-1/2" D
  - Carrier weight: 60lbs (27kg)
- Weight capacity:**
- 500lb maximum load

## TOOLS NEEDED

- Crescent or adjustable wrench
- 17mm wrench
- 24mm wrench
- 6mm Allen wrench

## EXPLODED VIEW & PARTS LIST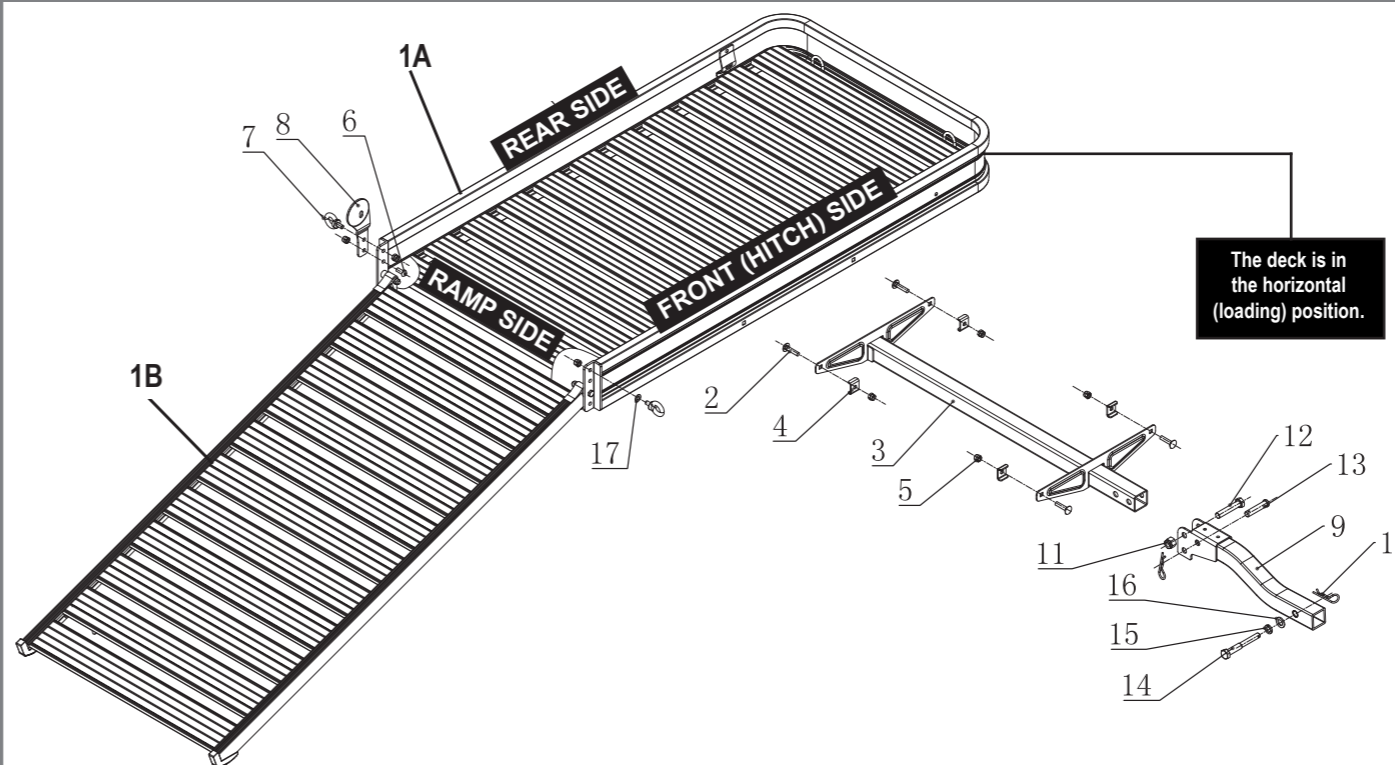



PART #	DESCRIPTION	QTY
1A	DECK	1
1B	RAMP	1
2	M10 Carriage Bolt	4
3	HITCH BAR SUPPORT	1
4	U Clamp	4
5	M10 Lock Nut	7
6	M10 Allen Head Bolt	1
7	Eye Bolt	2
8	Ramp Lock Plate	1
9	2" FOLDING HITCH BAR	1
10	Hitch Pin Clip	2
11	M16 Lock Nut	1
12	M16-80mm Bolt	1
13	5/8" Clevis Pin	1
14	M16-120mm Threaded Hitch Pin	1
15	M16 Lock Washer	1
16	M16 Flat Washer	1
17	M10 Flat Washer	1

## WARNINGS!

- ▲ Read these instructions carefully before installing, operating, or servicing the Mobility Carrier.
  - ▲ FAILURE TO FOLLOW THESE INSTRUCTIONS CAN LEAD TO SERIOUS INJURY!
  - ▲ THE MAXIMUM WEIGHT OF THE LOAD ON THE CARRIER MUST NOT EXCEED 500LBS (226 KG).
  - ▲ USE THE MOBILITY CARRIER ONLY WITH A PROPERLY INSTALLED 2" HITCH RECEIVER RATED TO SUPPORT THE COMBINED WEIGHT OF THE LOAD AND THE CARRIER.
- When calculating the weight resting on the hitch, make sure you include the weight of the carrier itself and do not exceed the manufacturer's specifications for your hitch.
- ▲ Distribute the load weight evenly on the platform.
  - ▲ This carrier is not intended for transporting humans or animals.
  - ▲ Do not carry loads longer or wider than the Mobility Carrier.
  - ▲ Do not carry flammable items on the Mobility Carrier.
  - ▲ Do not allow exhaust fumes to vent directly on loaded items. This is a FIRE HAZARD.
  - ▲ The manufacturer is not responsible for any damage if these warnings are disregarded.
  - ▲ Ensure the ramp is always locked into place securely before transporting a load.
  - ▲ When using the Mobility Carrier, ensure that your vehicle lights and signals are always visible in compliance with local laws and regulations.
  - ▲ Make sure you install the Mobility Carrier with the ramp on the passenger side of the vehicle.



## ASSEMBLING THE MOBILITY CARRIER

- As shown in Figure A — Attach the hitch bar support to the bottom of the deck by doing the following:
  - Place the deck (#1) on the ground upside down (rail faces down).
  - Align the four corners of the hitch bar support (#3) with the four mounting holes on the outside edge of the deck. The hitch bar end extends past the front (hitch) side of the deck (the side without reflectors).
  - Install the hitch bar support to the deck:
    - From the outside of the deck, insert an M10 carriage bolt (#2) through the hitch bar support mounting hole and into the corresponding deck mounting hole.
    - Slide a U clamp (#4) onto the end of the bolt and place the clamp onto the inside edge of the deck, then secure with an M10 lock nut (#5).
    - Repeat these steps for the three remaining mounting holes of the hitch bar support.
- As shown in Figure B — Install the ramp lock plate and eye bolts to the top of the deck by doing the following:
  - Turn the deck over so it is right-side up (rail faces up).
  - From the left corner of the deck (the side with reflectors), align the ramp lock plate (#8) to the top two mounting holes.
  - Install the ramp lock plate (#8) and eye bolts (#7) to the deck:
    - From the outside rear corner of the deck, insert the first eye bolt (#7) through the top hole in the ramp lock plate (#8) and into the top mounting hole on the deck, then secure it with an M10 lock nut (#5).
    - Beneath the eye bolt, from the inside of the deck, insert an M10 Allen head bolt (#6) through the mounting hole in the deck and through the ramp lock plate (#8), then secure it with an M10 lock nut (#5).
    - Slide an M10 washer (#17) onto the second eye bolt (#7), then from the outside front corner of the deck insert the bolt with washer through the top hole in the deck, and secure with an M10 lock nut (#5).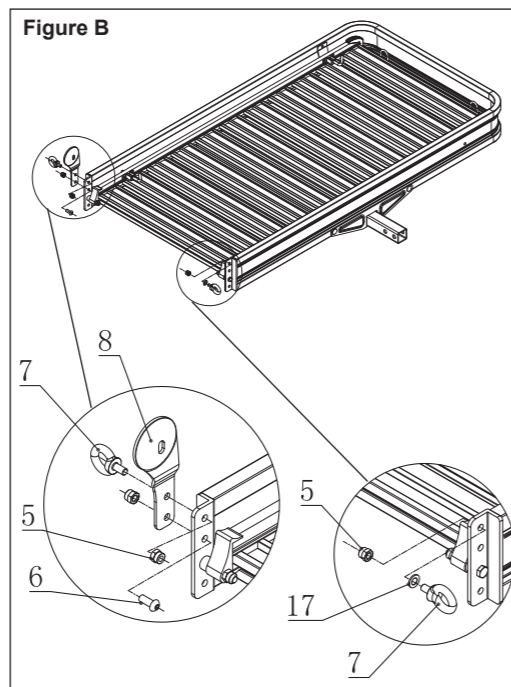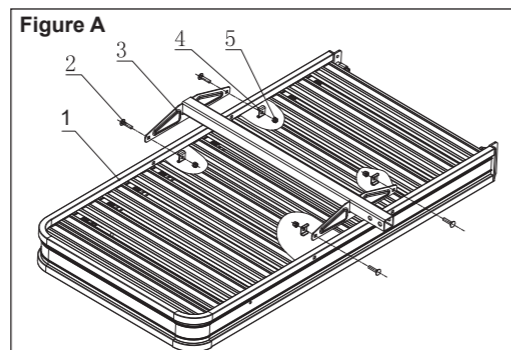

- As shown in Figure C — Attach the 2" folding hitch bar to the deck's hitch bar support by doing the following:
 



Slide the 2" folding hitch bar (#9) over the deck's hitch bar support (#3) and align the upper mounting hole on the end of the hitch bar to the mounting hole on the end of the hitch bar support, then secure with an M16 bolt (#12) and an M16 lock nut (#11).
- Pivot the Mobility Carrier from the horizontal (loading) position to the vertical upright (stowed) position in Figure D.
- As shown in Figure D — Attach the assembled Mobility Carrier to your vehicle's hitch receiver by doing the following:
  - ▲ WARNING: Get assistance before lifting the Mobility Carrier.
  - ▲ WARNING: Ensure that the ramp's lock pin (D-1) is fully seated to prevent the ramp from swinging out of position.
  - Insert the 5/8" clevis pin (#13) through the lower mounting hole on the hitch bar (Fig. D-2), then secure it with a hitch pin clip (#10).
  - With the Mobility Carrier in the vertical upright (stowed) position, lift and slide it into your vehicle's hitch receiver, and align it to the mounting holes on the receiver.
  - Place an M16 lock washer (#15) and M16 flat washer (#16) onto the M16 threaded hitch pin (#14), then insert the pin with washers into the mounting hole on the receiver and turn clockwise until seated, and secure with a hitch pin clip (#10).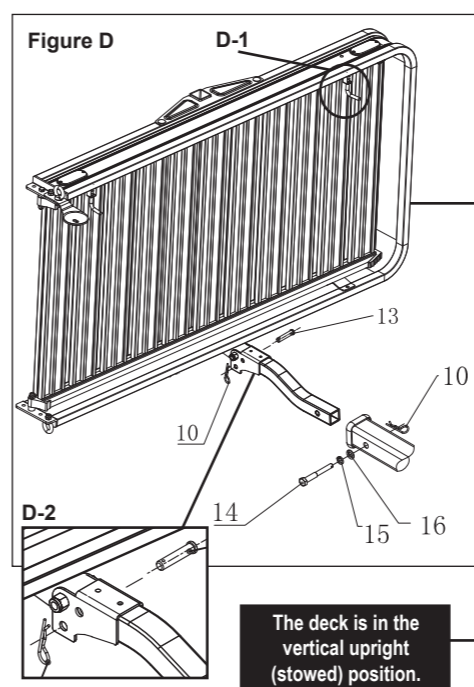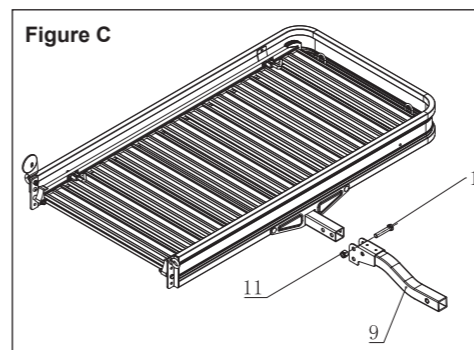

## USING THE MOBILITY CARRIER

### LOADING:

- To load the Mobility Carrier, adjust the deck to the horizontal (loading) position:
  - From a vertical upright (stowed) position, remove the clevis pin from the lower mounting hole (Fig. E-1).
  - Pivot the deck from the vertical upright (stowed) position to the horizontal (loading) position.
  - Insert the clevis pin into the top mounting hole (Fig. E-2) and secure with a hitch pin clip.
- Lower the ramp: Pull the lock pin handle as needed to release the ramp from the deck, then unfold and rotate the ramp to the ground.
  - ▲ CAUTION! Make sure the feet of the ramp rest flat on a solid, level surface during the loading and unloading of a wheelchair or scooter. Loading from a raised, flat, level surface, like a sidewalk, will lessen the angle of the ramp and make loading easier and safer.
- Load the carrier: With assistance, push the wheelchair or scooter up the ramp and onto the carrier bed. Make sure the load is centered on the carrier.
  - ▲ CAUTION! Do not ride on the wheelchair, scooter, or other vehicle that is being loaded onto, or unloaded from, the carrier.
- Secure the load: Set the brakes and secure the wheelchair or scooter in place using appropriate tie downs or ropes (sold separately). Fasten the tie downs or ropes to the tie down hooks on the carrier.
- Raise the ramp: Once the load is secure, rotate the ramp up from the ground to a vertical (loaded) position (Fig. F), and make sure the lock pin is securely seated in the lock pin plate.
  - ▲ CAUTION! Do not transport a load on the carrier without properly securing the load and locking the ramp to the carrier.

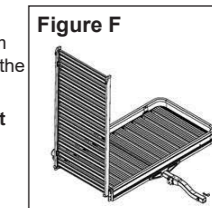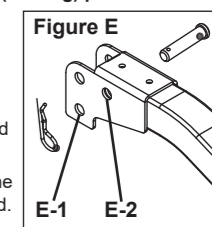### UNLOADING:

- To unload the carrier, unlock and lower the ramp, then remove the tie downs or ropes from the tie down hooks.
- Release the brakes, then, with assistance, carefully unload the wheelchair or scooter from the carrier.
  - ▲ CAUTION! Do not ride on the wheelchair, scooter, or other vehicle that is being loaded onto, or unloaded from, the carrier.
- Once the carrier has been unloaded, lift the ramp and fold it back into the deck, with the lock pin secured. Then return the carrier to the vertical upright (stowed) position.

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