ICE 
HORSE[®]
BETTER BY MILES

STIFLE WRAPS

WITH COLD CAPSULE[®] TECHNOLOGY

MADE IN

THE USA

CE

Thoroughly read the following information prior to application.
Product function requires proper application.

Instructions for freezing your reusable Cold Capsule® Inserts:

1. Place the Cold Capsule® inserts in the freezer with the round seal facing up. Try to keep uncovered so they can “breathe”. The formula will crystalize and become fluffy.
2. The initial freeze should be 6 hours or more. Refreezing after use will take 3-4 hours.
3. Remove the inserts from the freezer and gently massage to loosen snow crystals as shown in Fig. 1.
4. Cold Capsule® Inserts may be wiped clean with a damp cloth after use.
5. To refreeze, you can take the inserts out of the wraps saving freezer space, or place the entire wrap with the Cold Capsule® Inserts directly into your freezer.
6. If the inserts feel stiff, gently massage them or set out to thaw for a few minutes before using the wraps. When inserts become too chunky to conform to the horse's legs it's time to replace them.

**IF HEAT IS CALLED FOR,
USE DEEP HEAT™ by ICE HORSE®**

To Apply Stifle Wraps:

1. Wipe area to be cooled with a damp sponge. Skip this step if you are treating an open wound or an area that needs to be kept dry.
2. Separate the right and left wraps and attach together with suspender strap. Make the strap long to start as you can adjust strap length in the next steps.
3. Place the Cold Capsule® Inserts, Velcro® side down, on the inside of the Stifle Wraps as shown in Fig. 2.
4. Place strap over the rump of the horse slightly in front of the croup/rump as shown in Fig. 3.
5. Working one side at a time, place the wrap against the stifle area. Repeat the process on the other side and shorten/tighten the suspender strap as needed. Note: For ease in re-application, once the suspender strap is properly sized there is no need to take the wraps apart.
6. Check to see that the Cold Capsule® Inserts are secure against the stifle area and retighten the wraps as necessary.
7. Apply cold therapy for 30 minutes to 2 hours depending on condition or veterinarian recommendation.

**WHEN USING ICE HORSE® PRODUCTS, WE
RECOMMEND YOUR HORSE IS SECURELY TIED AND
NOT LEFT UNATTENDED.**

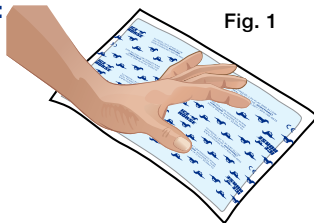
Fig. 1

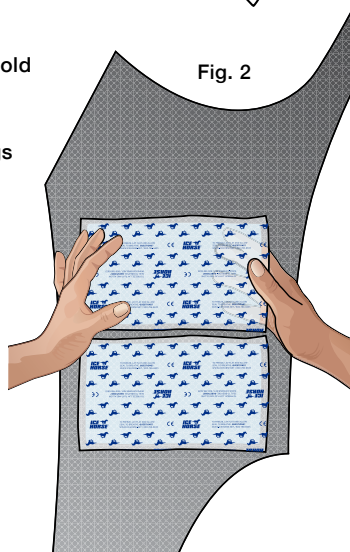
Fig. 2

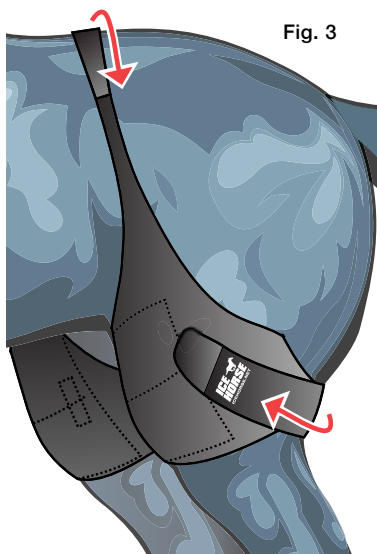
Fig. 3

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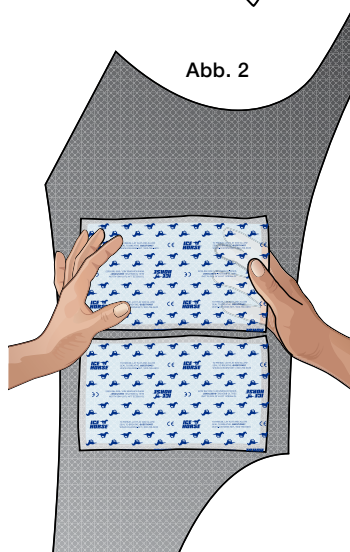
Abb. 2

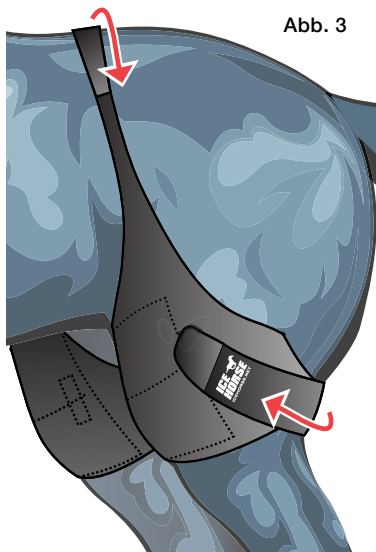
Abb. 3

WARNING: THIS PRODUCT IS ONLY TO BE USED AS DIRECTED. OBSERVE SKIN CONDITION DURING USE. IF REDNESS, PAIN OR SWELLING DEVELOPS, OR CHANGE IN SKIN SENSITIVITY OCCURS, DISCONTINUE USE AND CALL YOUR VETERINARIAN. BE AWARE THAT NERVE IRRITATION OR MUSCULAR REACTION MIGHT BE ASSOCIATED WITH INCREASED SKIN SENSITIVITY AND IRRITATION. DISCONTINUE THERAPY UNTIL THE EXACT CAUSE OF THE PROBLEM CAN BE DETERMINED.

CAUTION: Cold Capsule® Inserts should not burn the hair or hide of an animal but direct application of any non-coated cold / heat pack directly to human skin is not recommended. Acceptable and recommended application for human use is over a thin dressing. Cold Capsule® Inserts are made of a non-toxic formula and are not intended for internal use or consumption.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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