
FULL HIND LEG WRAPS & SUSPENDERS

WITH COLD CAPSULE® TECHNOLOGY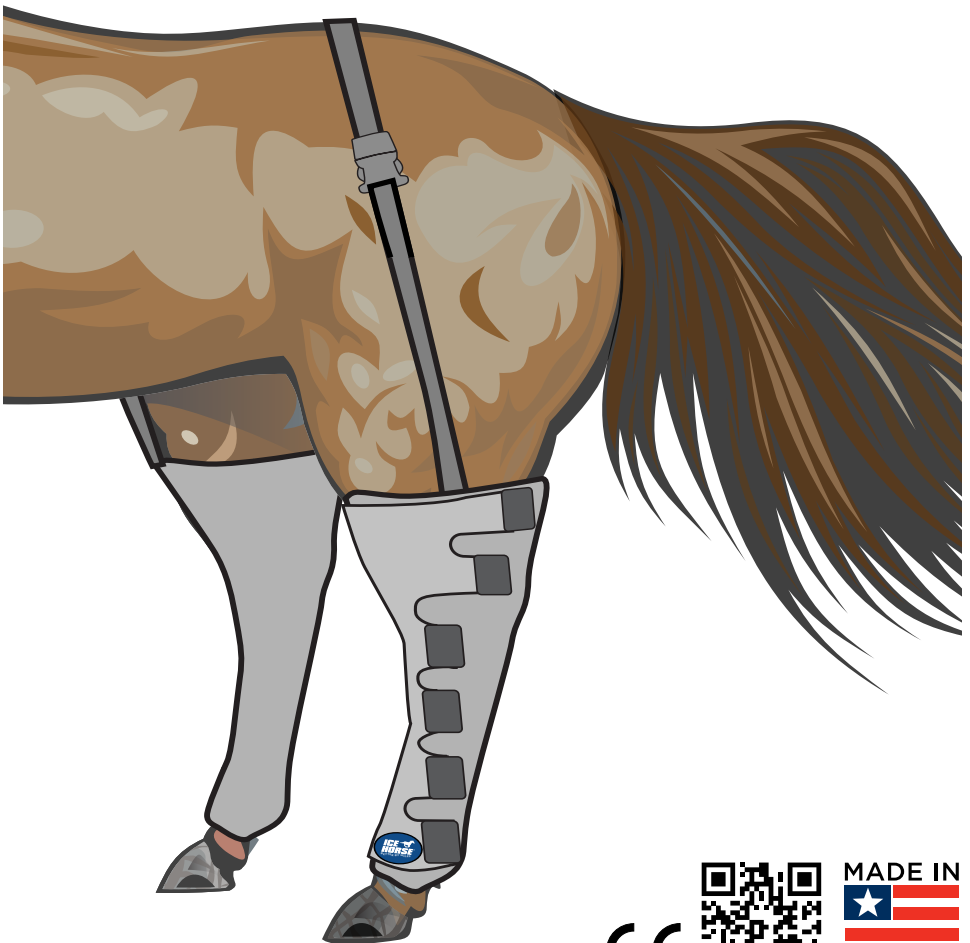

Thoroughly read the following information prior to application.
Product function requires proper application.

Instructions for freezing your reusable Cold Capsule® Inserts:

1. Place the Cold Capsule® inserts in the freezer with the round seal facing up. Try to keep uncovered so they can “breathe”. The formula will crystalize and become fluffy.
2. The initial freeze should be 6 hours or more. Refreezing after use will take 3-4 hours.
3. Remove the inserts from the freezer and gently massage to loosen snow crystals as shown in Fig. 1.
4. Cold Capsule® Inserts may be wiped clean with a damp cloth after use.
5. To refreeze, you can take the inserts out of the wraps saving freezer space, or place the entire wrap with the Cold Capsule® Inserts directly into your freezer.
6. If the inserts feel stiff, gently massage them or set out to thaw for a few minutes before using the wraps. When inserts become too chunky to conform to the horse’s legs it’s time to replace them.

**IF HEAT IS CALLED FOR,
USE DEEP HEAT® by ICE HORSE®**

Instructions for Wrap:

1. Place the Cold Capsule® Inserts along the Full Hind Leg Wrap as shown in Fig. 2.
2. Wipe area to be cooled with a damp sponge. Skip this step if you are treating an open wound or an area that needs to be kept dry.
3. Place the Wrap on the horse’s leg, securing the inside sleeve tightly against the horse as shown in Fig. 3.
4. Wrap the outer compression sleeve around the leg and secure tightly with the Velcro® straps starting with the middle straps. Next tighten the bottom straps and finish at the top as shown in Fig. 4.
5. Place suspender strap over the rump/croup of the horse and attach on the outside of each leg wrap to keep the ice boots from slipping down.
6. Check to see that the ice is secure against the leg and retighten wraps as necessary.
7. Apply cold therapy for 30 minutes to 2 hours depending on condition or veterinarian recommendation.

**WHEN USING ICE HORSE® PRODUCTS, WE RECOMMEND
YOUR HORSE IS SECURELY TIED AND NOT LEFT
UNATTENDED.**

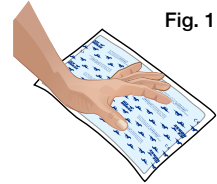
Fig. 1

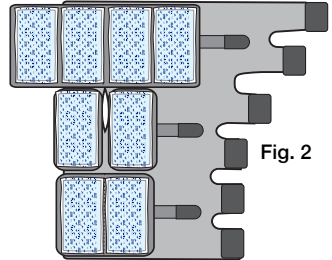
Fig. 2

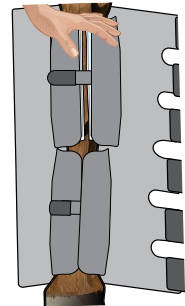
Fig. 3

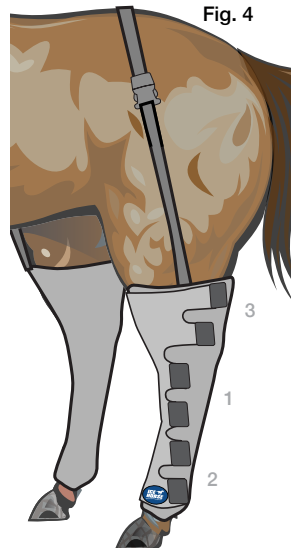
Fig. 4

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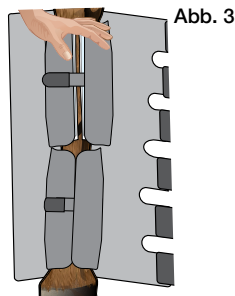
Abb. 3

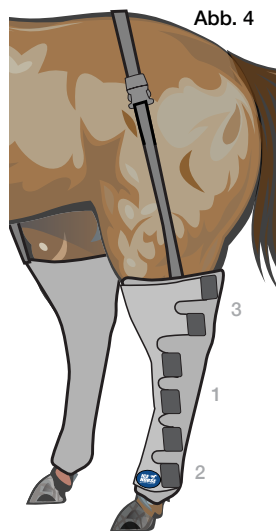
Abb. 4

WARNING: THIS PRODUCT IS ONLY TO BE USED AS DIRECTED. OBSERVE SKIN CONDITION DURING USE. IF REDNESS, PAIN OR SWELLING DEVELOPS, OR CHANGE IN SKIN SENSITIVITY OCCURS, DISCONTINUE USE AND CALL YOUR VETERINARIAN. BE AWARE THAT NERVE IRRITATION OR MUSCULAR REACTION MIGHT BE ASSOCIATED WITH INCREASED SKIN SENSITIVITY AND IRRITATION. DISCONTINUE THERAPY UNTIL THE EXACT CAUSE OF THE PROBLEM CAN BE DETERMINED.

CAUTION: Cold Capsule® Inserts should not burn the hair or hide of an animal but direct application of any non-coated cold / heat pack directly to human skin is not recommended. Acceptable and recommended application for human use is over a thin dressing. Cold Capsule® Inserts are made of a non-toxic formula and are not intended for internal use or consumption.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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