HOOF BOOT

WITH COLD CAPSULE® TECHNOLOGY

Thoroughly read the following information prior to application.
Product function requires proper application.

Instructions for freezing your reusable Cold Capsule® Inserts:

1. Place the Cold Capsule® inserts in the freezer with the round seal facing up. Try to keep uncovered so they can “breathe”. The formula will crystalize and become fluffy.
2. The initial freeze should be 6 hours or more. Refreezing after use will take 3-4 hours.
3. Remove the inserts from the freezer and gently massage to loosen snow crystals as shown in Fig. 1.
4. Cold Capsule® Inserts may be wiped clean with a damp cloth after use.
5. To refreeze, you can take the inserts out of the wraps saving freezer space, or place the entire wrap with the Cold Capsule® Inserts directly into your freezer.
6. If the inserts feel stiff, gently massage them or set out to thaw for a few minutes before using the wraps. When inserts become too chunky to conform to the horse's legs it's time to replace them.

IF HEAT IS CALLED FOR, USE **DEEP HEAT®** INSERTS
by ICE HORSE®

Application instructions for Hoof Boot

- Open up the boot and place the Cold Capsule® inserts on the inner pads of the boot as shown in Fig. 2.
- Lift the horse's hoof and place the boot on it. Note: The front of the boot has the Ice Horse® logo as shown in Fig. 3.
- Set the hoof down and wrap the inner pads with the inserts around the horse's hoof pressing then against the hoof as tightly as possible as shown in Fig. 4.
- Pull the rear flap around the heel and back of hoof/leg and fasten against the inner pad., stretching the wrap for a secure fit as shown in Fig. 5.
- Pull the front wrap around the hoof stretching the wrap for a secure fit, and fasten to the rear wrap. Note: the inner pads and entire wrap is compatible with the Velcro® for ease in in adjustment and secure fit as shown in Fig. 6.
- Apply the cold therapy treatment for 30 minutes 2 hours depending on condition or recommendation veterinarian.

WHEN USING ICE HORSE® PRODUCTS, WE RECOMMEND YOUR HORSE IS SECURELY TIED AND NOT LEFT UNATTENDED.

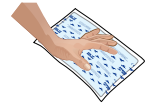
Fig. 1

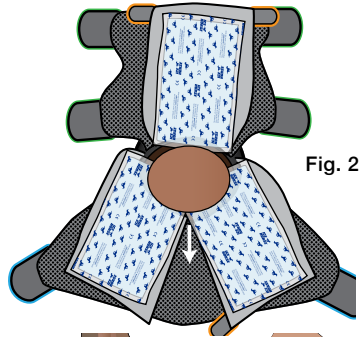
Fig. 2

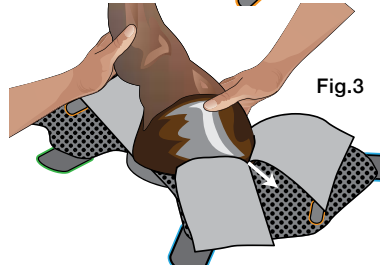
Fig. 3

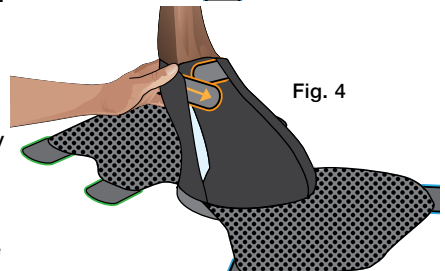
Fig. 4

Fig. 5

Fig. 6

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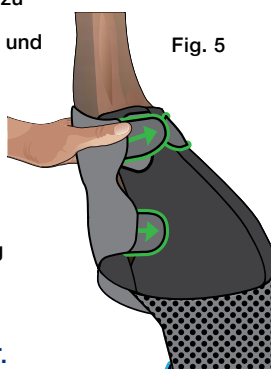
Fig. 5

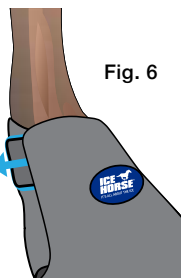
Fig. 6

WARNING: THIS PRODUCT IS ONLY TO BE USED AS DIRECTED. OBSERVE SKIN CONDITION DURING USE. IF REDNESS, PAIN OR SWELLING DEVELOPS, OR CHANGE IN SKIN SENSITIVITY OCCURS, DISCONTINUE USE AND CALL YOUR VETERINARIAN. BE AWARE THAT NERVE IRRITATION OR MUSCULAR REACTION MIGHT BE ASSOCIATED WITH INCREASED SKIN SENSITIVITY AND IRRITATION. DISCONTINUE THERAPY UNTIL THE EXACT CAUSE OF THE PROBLEM CAN BE DETERMINED.

CAUTION: Cold Capsule® Inserts should not burn the hair or hide of an animal but direct application of any non-coated cold / heat pack directly to human skin is not recommended. Acceptable and recommended application for human use is over a thin dressing. Cold Capsule® Inserts are made of a non-toxic formula and are not intended for internal use or consumption.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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