# DEEP HEAT<sup>®</sup> BY ICE HORSE<sup>®</sup>

MADE IN  
  
THE USA

Thoroughly read the following information prior to application.  
Product function requires proper application.

**Always bring bag to room temperature before heating.**

#### MICROWAVE

1. After bringing to room temperature, lay bag facedown with seam up in microwave.
2. Heat at full power as shown: Single bag instructions.

WATTAGE	1000	800	600	UNSURE
HEATING TIME	35 seconds	45 seconds	60 seconds	35 seconds

3. Remove from microwave and knead bag for one minute to allow gel to heat evenly (temperature will continue to increase slightly).
4. Test for desired temperature. If too hot, allow to cool. If additional heat is desired, return to microwave and heat in 10 second cycles.
5. When desired temperature is achieved, place bag onto ICE HORSE® wrap with Velcro®, then apply to desired area.

#### HOT WATER

1. Bring bag to room temperature.
2. Bring at least 3 quarts of water to a rolling boil.
3. Remove water from heat source. Gently immerse bag for a maximum of 5 minutes.
4. Remove bag from water, place bag into ICE HORSE® wrap pocket, then apply to desired area.

**DISCARD IF PUNCTURED OR TORN ALWAYS PLACE IN WRAP SLEEVE, OR ATTACH TO WRAP WITH VELCRO® BEFORE APPLICATION.**

- Repeat this cycle as prescribed.
- Follow instructions from your veterinarian for best placement and duration.

**WHEN USING ICE HORSE® PRODUCTS, WE RECOMMEND YOUR HORSE IS SECURELY TIED AND NOT LEFT UNATTENDED.**

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---

**WARNING: THIS PRODUCT IS ONLY TO BE USED AS DIRECTED. OBSERVE SKIN CONDITION DURING USE. IF REDNESS, PAIN OR SWELLING DEVELOPS, OR CHANGE IN SKIN SENSITIVITY OCCURS, DISCONTINUE USE AND CALL YOUR VETERINARIAN. BE AWARE THAT NERVE IRRITATION OR MUSCULAR REACTION MIGHT BE ASSOCIATED WITH INCREASED SKIN SENSITIVITY AND IRRITATION. DISCONTINUE THERAPY UNTIL THE EXACT CAUSE OF THE PROBLEM CAN BE DETERMINED.**

---

CAUTION: Cold Capsule® Inserts should not burn the hair or hide of an animal but direct application of any non-coated cold / heat pack directly to human skin is not recommended. Acceptable and recommended application for human use is over a thin dressing. Cold Capsule® Inserts are made of a non-toxic formula and are not intended for internal use or consumption.

---

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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