



zenpet™



# E-Clear

Neck Size:  
9.5-15.5 in  
(24.1-39.4 cm)

## Recovery Collar



For post surgery, hot spots,  
medication & wound healing



Padded comfort band



Adjustable custom fit

## 🐾 Directions 🐾

1. Place E-Collar around pet's neck.
2. Line up velcro tabs to the desired matching slits on opposite end of collar.
3. Thread velcro tab down first slit & back up second slit.
4. Fold velcro back over top to fasten.
5. Repeat 1-2 more times to secure all straps, depending on E-Clear size.
6. Attach the E-Collar to your pet's regular collar with gauze or string.

**Fit note:** E-Collar should extend beyond the length of our pet's nose. Make sure the E-Collar fits snugly around your pet's neck with enough room to fit two fingers between it and your pet.



**CAUTION: FOR PET USE ONLY.** Intended to be used for a short period of time and under supervision, Carefully monitor pet for the first several hours of use to ensure pet has become accustomed to wearing the collar. Inspect regularly and replace if any part becomes damaged.

CS Tech US  
Oceanside, CA  
[www.zenpetusa.com](http://www.zenpetusa.com)  
1-844-4ZENPET  
Assembled in Mexico  
with US materials



Patent Pending